

Surgicut Soft Tissue Trimmer

Date of issue: 06.11.2014

Last revision date: 21.11.2017



**Example*

Application:

Cutting of oral soft tissue without or with minimal bleeding. To be used only in dental FG handpieces.

Mode of application:

For reprocessing (cleaning, disinfection and sterilization) see the separate instructions for reprocessing.

Before each use make sure the tip of the instrument is intact. For best results please observe the recommended speeds of 30.000 to 300.000 rpm at all times. Water cooling needs to be avoided so that the blood coagulation effect of the instrument can be effective. Insert the instrument into the mouth only when the desired rotating speed has been reached. Only use within the limits of the instructions for use. Please observe the instructions for use and recommendations of the handpiece/turbine manufacturer.

Please note our additional general instructions for the use of rotary instruments (www.diaswiss.ch).

Warnings:

Avoid using the instrument as a lever and jamming (risk of fracture). Pressure and speed depend on the oral situation of the mucosa. Also use as per instructions for use. Please observe the instructions for use and recommendations of the handpiece/turbine manufacturer. Do not store the instrument in plastic bags. Ensure dry storage at ambient temperature. Instruments need to be sterilized before disposal in order to avoid transmission of infectious diseases.

Quality:

Blanks made of corrosion-resistant medical-grade hardened steel. Zirconia stabilized with yttrium. (ZrO₂)

Risks:

No risks have been observed when used according to instructions for use.

Disposal:

Instruments to be discarded must be sterilized before their disposal with general clinical waste.

CE 1250

Rev. 11/17



DIASWISS S.A.
Rte de St Cergue 293
CH-1260 Nyon
Switzerland